


How to Register and Use the Polymer Detection Kit

1. Open the kit and use the barcode scanner to scan the 2 registration barcodes inside the box separately.

NOTE:

- The two barcodes can be scanned in any order.
- A test kit only needs to be registered the first time it is used, and no re-registration is required for subsequent uses.

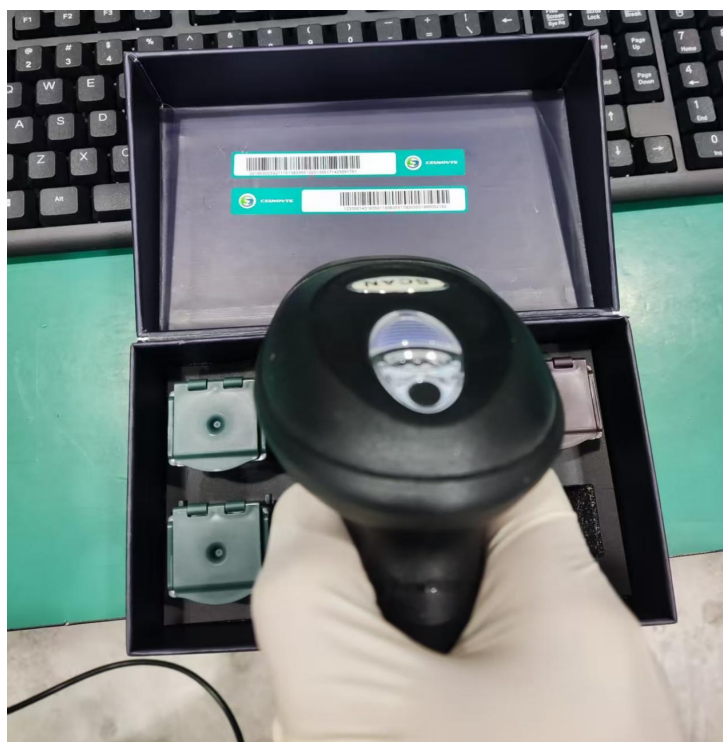

Reagent Barcode Verification

Enter Reagent ID and Click "Verification"

For reagent system, firstly verify the first ID and then enter the sencond.

Name: 单染聚合物检测系统 (SD5300)

Catalog Number: 0

UPI: 35513221

Batch Number: 51383

Validity Date: 2027-03-16

Capacity(ul): 30000

Position	UPI	Reagent
1	35513221	*Peroxidase Block Reagent
2	35517142	*Post Linker
3	35517513	*Secondary Antibody
4	35514318	*DAB - A
5	35511806	*DAB - B
6	35517930	*DAB - B
7	35519660	*Hematoxylin

OK Cancel

2. Place the reagent bottles on the reagent rack with the following requirements:

- The 7 reagent bottles should be placed closely together, and their front-to-back order does not matter.
- Reagents must be placed at the front end of the reagent rack.
- There should be no gaps between the 7 bottles.



3. Open the lips of all reagent bottles.



4. Push the reagent strip into the reagent compartment of the instrument.

